
 University Health™	Policy #: Rad Proc 14. 15. 6
SUBJECT: GYN PELVIS 2 (SMALL)	(SMALL)Effective: 10/1/2013 Revised: 2/2015: 10/2017
APPROVED BY Eduardo Gonzalez-Toledo, MD PhD	Page 1 of 2

ORIENTATION: FEET FIRST/SUPINE COIL: HD BODYLOWER

PLANE	3 PLN LOC	AX T2 FS	SAG T2	AX T1	AX T1 FS	Ax T1 FS C	SAG T1 FS C+	AX LAVA C
SEQ	S ECHO	FSE-XL	FSEXL	FSE-XL	FSEXL	FSE-XL	FSEXL	LAVA
MODE	2D	2D	2D	2D	2D	2D	2D	3D
IMAGING OPTIONS	SEQ/FAST/SS	FC/NPW/EDR/TRF/FAST	FC/NPW/EDR/TRF/FAST	FC/NPW/EDR/FAST	FC/NPW/EDR/FAST	FC/NPW/EDR/TRF/FAST	FC/NPW/EDR/FAST	Fast/aip2/As set
TE	80 ms+	90 ms +	102 ms +	MIN FULL	MIN FULL	MIN FULL	MIN	
TR	MIN	4234 ms +	3934 ms +	450 ms	567.0ms	567.	650 ms	
TI								
FLIP ANGLE								12
ETL		21	21	3	2	4	4	
BW	83.33	15.63	35.25	31.25	27.78	27.78	25	
FOV	44	22	22	22	22	22	25	30
SLICE THICKNESS	8	4	4	4	4	4	4	3
SLICE SPACING	10	1	1	1	1	1	1	
Frequency	320	320	320	320	256	256	256	288
Phase	192	224	224	224	192	192	192	192
NEX		3	2	2	2	3	2	
PHASE FOV	1	1		1	1		1	
FREQ DIR	UNSWAP	Freq	S/I	R/L	R/L	R/L	R/L	

 University Health™	Policy #: Rad Proc 14. 15. 6
SUBJECT: GYN PELVIS 2 (SMALL)	(SMALL)Effective: 10/1/2013 Revised: 2/2015: 10/2017
APPROVED BY Eduardo Gonzalez-Toledo, MD PhD	Page 1 of 2

ORIENTATION: FEET FIRST/SUPINE COIL: HD BODYLOWER

FLOW COMP DIR		FREQ	FREQ	FREQ	FREQ	FREQ	FREQ	
SHIM	AUTO	AUTO	AUTO	AUTO	AUTO	AUTO	AUTO	AUTO
PHASE CORRECT	OFF	ON	ON	ON	ON	ON	ON	OFF

EXAM: SMALL FOV GYN PELVIS (GYN 2) ORIENTATION: FEET FIRST/SUPINE COIL: HD BODYLOWER

NOTES: Call the Radiologist to review the images before administering contrast. Make sure that the pre and post axial images are in the same location. Use the small FOV pelvis protocol when a urethral diverticulum or bladder pathology is indicated in the hx., or as directed by the Radiologist.

AX T1 FS PRE and POST: The mask will be acquired first, then the scanner will pause and the green light will appear. After the table is removed from the magnet and then sent back within the magnet, the green light will disappear. Upon returning to the control room, click on prep scan and the green light will reappear. Then start the scan. Send the mask, each phase, and the subtractions for each phase to PACS.

Utilize Dotarem for this exam, Use 0.2 mL/kg body weight for age 2 and older per Dr. Sangster; under age 2 consult Radiologist