

Note: Additional sequences may be requested at the discretion of the Radiologist monitoring the exam.

EXAM: SHOULDER ARTHROGRAM PRE

ORIENTATION: HEAD FIRST/SUPINE

COIL: SHLDRPA4

PLANE	3 PLN LOC	COR T2 FS	COR T2	AX GRE T2*
SEQ	GRE	FSE XL	FSE XL	GRE
MODE	2D	2D	2D	2D
IMAGING OPTIONS	SEQ/FAST	NPW/TRF/FAST	NPW/TRF/FAST	NPW/EDR/ZIP512
TE		90 ms +	90 ms +	20
TR		3800 ms +	3800 ms +	500
FLIP ANGLE				20
ETL		12-18	12-18	
BW		31.25	31.25	20.83
FOV	20	16	16	16
SLICE THICKNESS	7	4	4	4
SLICE SPACING	5	0.5	0.5	0.5
Frequency	256	256	320	320
Phase	128	192	224	224
NEX	1	4	4	3
PHASE FOV	1			
FREQ DIR	UNSWAP	S/I	S/I	S/I
FLOW COMP DIR				
SHIM	AUTO	AUTO	AUTO	AUTO
PHASE CORRECT	OFF	ON	ON	OFF

Written: 10/2013 Reviewed: 05/2014; 02/2015