

Note: Additional sequences may be requested at the discretion of the Radiologist monitoring the exam.

EXAM: SHOULDER ARTHROGRAM POST ORIENTATION: HEAD FIRST/SUPINE COIL: SHLDRPA4

PLANE	3 PLN LOC	COR T1 FS C+	SAG T1 FS C+	AX T1 FS C+	AX T1 C+
SEQ	GRE	FSE XL	FSE XL	FSE XL	FSE XL
MODE	2D	2D	2D	2D	2D
IMAGING OPTIONS	SEQ/FAST	NPW/TRF/FAST	NPW/TRF/FAST	NPW/TRF/FAST	NPW/TRF/FAST
TE		MIN FULL	MIN FULL	MIN FULL	MIN FULL
TR		400-600 ms	400-600 ms	400-600 ms	400-600 ms
FLIP ANGLE					
ETL		2	2	2	2
BW		20.83	20.83	20.83	20.83
FOV	20	16	16	16	16
SLICE THICKNESS	7	4	4	4	4
SLICE SPACING	5	0.5	0.5	0.5	0.5
Frequency	256	256	256	256	320
Phase	128	192	192	192	224
NEX	1	3	3	3	3
PHASE FOV	1				
FREQ DIR	UNSWAP	S/I	S/I	S/I	S/I
FLOW COMP DIR					
SHIM	AUTO	AUTO	AUTO	AUTO	AUTO
PHASE CORRECT	OFF	ON	ON	ON	ON

Written: 10/2013 Revised: 01/2009; 06/2013 Reviewed: 05/2014; 02/2015; 04/2016